Instructions for adaptation

X-1714 (TLSO mujer)
X-1742 (TLSO hombre)
X-1541 (LSO mujer)
X-1542 (LSO hombre)

T.L.S.O.

L.S.O.

RECTUS

2010

SN

LOT

1- Adaptación del TLSO RECTUS

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1- TLSO RECTUS adaptation

Before adapting the product to the patient, you should confirm that the following components are available:

S-174...-... -AA: Aregaten TLSO's Anterior Valve, with *Texton Control System* support piece.

S-174...-... -AP: Aregaten TLSO's Posterior Valve.

S-174...-... -BA: TLSO's Anterior Padding.

S-174...-... -BP: TLSO's Posterior Padding.

S-1718-C...: TLSO's Sternum Piece.

S-1940-00...-BC: TLSO's Sternum Piece Padding.

S-1718-000-AL: Black Alligator 38 mm (8 units).

Place the paddings in the corresponding valves and in the sternum piece.

01 Place the patient sideways avoiding sudden movements.

02 Rotate the patient forwards and locate the waist, so the anatomic shape of the posterior valve can be properly placed.

03 Hold the posterior valve and rotate the patient, placing him face upwards so the piece fits in place.

04 Place the anterior valve on the patient and adjust both valves with the middle Velcro fasteners, pulling them both simultaneously.

05 Check the length of the Velcro fasteners, it's possible that you need to cut them so they can adjust to the patient's measures.

06 Once the fastener strip is adjusted, place the Alligator in the extremity in order for the male to stay in the correct position when the fastener passes the buckler.

07 Proceed to adjust the fasteners, always pulling both sides simultaneously in order to achieve a correct adjustment.

08 Rotate the piece of the Texton Control System to place the sternum part.

Adjust the elastic strip fasteners once the alligators are placed, strain them simultaneously.

The Rectus TLSO is placed and adapted, if any modification is to be made to one of the Aregaten made plastic valves, the padding should be removed, apply heat with a hot air gun until the plastic can be mouldable. Place the padding again and adapt it to the patient.

Note: if the patient is standing, the instructions to place the TLSO are the same.

ENGLISH

RECTUS LSO adaptation

Before adapting the product to the patient, you should confirm that the following components are available:

- S-154...-... -AA:** LSO's Aregaten Anterior Valve.
- S-154...-... -AP:** LSO's Aregaten Posterior Valve.
- S-154...-... -BA:** LSO's Anterior Padding
- S-154...-... -BP:** LSO's Posterior Padding
- S-1718-000-AL:** Black Alligator 38 mm. (6 units).

To place the LSO follow the TLSO's instructions ignoring the ones regarding the sternum piece.

LSO placed in the patient.

Remember that both the Rectus TLSO and LSO can be made to measure, following the Epsilon protocol. Besides the pendulous abdomen option is available.

For further information visit our web site ()

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