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Set and adjust the corset (brace) on the patient using velcro strips.

Check the length of the velcro strips, in case there is need to cut.

To shorten the velcro strips, peel off the male Alligator from the female velcro strip.

Put the corset on the patient and mark the excess of the flap at the top


Cut off the excess of the female velcro strip.

Stick again the male Alligator on the female velcro strip.


After adjusting the velcro, mark with a pen those areas where you want to cut.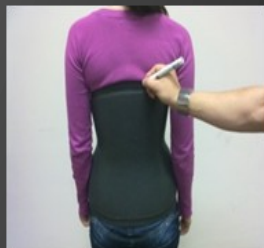

Mark the corset (brace) at the back if it's necessary to make some cuts.

For putting the inner flap setting velcro, center the flap over the corset (brace) and mark down.

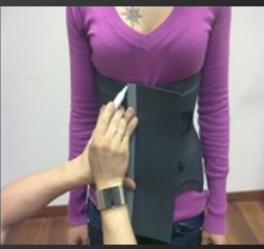Make another mark up. Now we have marked the position of the adhesive velcro, which will be on the inside and will set the flap to the corset.

Overlap the adhesive velcro strip, following the marks to ensure a correct position.

Remove the white strip and stick the adhesive velcro so that the black female velcro is facing the body.


Make sure the velcro is stuck and there are no air bubbles.

Stick the flap on the black inside velcro.

Mark the excess of the flap at the bottom.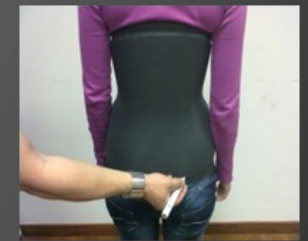

After marking, remove the corset and cut along the marks with scissors.

The corset (brace) is adapted and completed.

If it's necessary to make any changes in the rigid inner structure, peel off the *Multiform* with alcohol.

Heat the *Subortoiben* using a heat gun, until you can shape.

Finally, paste the *Multiform* with glue or double-sided adhesive tape.

Remember that we have special options:

- Prominent abdomen
- 0° lordosis
- Prominent ribs
- **Apertura cresta iliaca??????????**