# RECTUS

Ref. X-1240

**ORTESIS DE SOPORTE LUMBAR  
LUMBAR ORTHOSIS  
CEINTURE DE SOUTIEN LOMBAIRE  
LUMBALSTÜTZORTHESE  
ORTÓTESE LOMBAR**

**INSTRUCCIONES DE USO Y CONSERVACION  
APPLICATION INSTRUCTIONS, USE AND CARE  
INSTRUCTIONS, UTILISATION ET ENTRETIEN  
GEBRAUCHS UND PFLEGEANLEITUNG  
INSTRUÇÕES DE USO Y CONSERVAÇÃO**

Ortoiberica S.L.  
Parque Tecnológico Asturias parcela 1  
33428 Llanera (Principado de Asturias)  
España  
tfno.: 34 985794800 fax: 34 985794810  
mail: [welcome@ortoiberica.es](mailto:welcome@ortoiberica.es)  
[www.ortoiberica.es](http://www.ortoiberica.es)

Licencia Sanitaria P.S. 67

Estimado/a paciente:

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Dear Patient,

We appreciate your trust in our company, to choose to use the lumbar orthosis Rectus. Before putting the orthosis, please read the operating instructions that follow. In case of doubt contact your doctor, your orthopedics, our web site ([www.rectus.es](http://www.rectus.es)), or with our customer support department. We reinforced our appreciation for your trust, desiring a quick improvement of their health status.

**INDICATIONS:** Back pain, pre and post-surgery treatments, degenerative processes, muscle imbalance of the abdomen/spine, mild disc degeneration.

**INSTRUCTIONS:** Initially the orthosis should always be placed by an orthopedic technician or other medical professional.

To achieve effective results, the size should be selected appropriate to the patient (see the equivalence of sizes on the page 7 of this manual). An excessive compression and a little one can be harmful, so it is essential that the user set correctly the orthosis.

The patient must put the orthosis on the lumbosacral area, trying to adjust it in the correct area. (See Figure 1 on page 6). The orthosis are composed of four elastic bands, where exists some hand support bands that allow the patient to put the fingers to facilitate the placement of the orthosis. One of these hand support band has a grey colour, and the other three are dark red. Enter your fingers on the hand support bands of the lower bands (Figure 2), adjusting and focusing properly on the sacral lumbar area. When the orthosis is in position (Figure 3) fix first the grey hand support band. Then do the same with the dark red hand support band (Figure 4), correctly adjusting the compression. Once properly located the two bands, do the same with the other two bands (Figure 5). It is indifferent the order of placing them. The bands can be shifted down in order to avoid an excessive compression at the diaphragm, making breathing easier.

**PRECAUTIONS:**

The use of the orthosis should always follow the instructions of the prescriber and her adaptation must be tailored for each patient, always accompanied by an orthopedic professional or a qualified person. Avoid applying too much pressure with velcro, especially after meals.

So far there are known side effects, however, if you notice any irritation in the area of application of the orthosis, consult your doctor. The combination of lumbar orthosis Rectus with other products or orthoses should be consulted and approved by the prescriber. In case of excessive sweating, we recommend the use of orthosis with cotton shirts. Should not be used during sleep, and should be relief the pressure during the day.

In preparation of the orthosis were used flammable, so you should not expose the product to any source of combustion. Do not expose your product to any heat source that deteriorates the shape or the materials that constitute it.

**WARNINGS:**

The composition of materials is detailed in the orthosis itself. When the orthosis is not being used, it should fold it properly and store it in the package. To discard the product or the packaging, follow the directions laid down by the authorities. Only recommends using the same orthosis for a single patient. Avoid share it with another user, even if he has the same diseases.

**RECOMMENDATIONS FOR MAINTENANCE AND CLEANING:**

Always close the velcro for cleaning, fixing them together. The orthosis should be washed regularly with mild soap for delicate clothes, with water at 30 ° C. Use a dry towel or a dry cloth to absorb humidity and let dry at room temperature. Never use the drying machine our an heat source (or solar) to accelerate the drying of the product. Detergents can cause irritation, so you should rinse properly the orthosis.

Never use alcohol, stain removers or other material to remove stains.

**WARRANTY:**

Ortoibérica SL assures their products according to the laws, where it has not been damaged the initial configuration of the product. Ortoibérica does not ensure that the product has not been used properly or whose initial configuration has been modified.

For any claim is essential to state the number of the PARCEL that are present in the orthosis, which coincides with what comes in the package. If the number is unreadable, even for his own use, Ortoibérica not accept any complaints.

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